

# HANDYVAP VAP 01

STEAM INHALER

## Making quick relief, easier.



For over a hundred years, steam has been a commonly used therapy for relieving and treating congestion, sinusitis, cold, and ailments of the nose and throat. With steam inhalers that are easy to use, doctors still recommend steam for nose and throat ailments. It is the quickest way to relieve congestion.

**Make quick relief easy with Medtech's Handyvap.**

### KEY FEATURES



100% Plastic Body  
to prevent shocks.



1.5 meter Cord



Double Wall Body



Special Lock



Safety Cap

### BENEFITS



Avoids Burns



Avoids Accidental  
Opening



Avoids Injuries



### Useful For -

Steam inhalation is a proven measure of treatment for colds, bronchitis, sinus, asthma, arthritis, laryngitis, aromatherapy and ayurvedic treatments.

## HOW TO USE

01

Unscrew the cap by rotating it anti-clockwise.

02

Fill the bottle halfway and screw on the cap. Rotate clockwise to close.

03

Put the plug back in the socket. Switch the power on and wait.

04

Once the vapour is uniform, inhale the vapour with your nose.

05

On finishing inhalation, switch the power off.

\* Make sure the water is not below minimum.

## TIPS FOR SAFETY



Place the unit upright while in use or filled with hot water.



Do not leave unit unattended around children.



Keep it away from water. Do not plug-in a wet pin.



Do not pull the wire to unplug or switch off. Switch off the button & then pull the plug.

Note: The information provided is best to our knowledge and subject to change without notice.

Picture for illustration purposes only, actual product and accessories may vary.

Specifications are subject to change without notice.

### OUR PRODUCT RANGE

Nebulizer | BP Monitor (Digital/LED) | Steam Inhaler | Digital Thermometer | Pulse Oxymeter | Oxygen Concentrator  
Air Pressure Mattress | Auto CPAP | Needle Burner & Syringe Destroyer | Bipap | Infrared Thermometer

## MEDTECH LIFE PVT. LTD. (Formerly Nulife Global Medical Devices Pvt. Ltd.)

B6, Byculla Service Industries, D.K. Marg, Byculla, Mumbai 400 027, India.

+9122 2374 8372 / 73 +9122 6657 8989 info@medtechlife.com

Domestic: +91 82917 26885 / +91 82917 26884 / 83 Export: +91 91369 23382 / +91 93263 56572

<https://bit.ly/2NYdqh3>

<https://www.facebook.com/medtech.India>

[medtechlife](https://www.youtube.com/medtechlife)