

HANDY TMP 03

DIGITAL THERMOMETER

Monitor your
temperature.
Manage illness.



A temperature rise is generally caused due to an infection.
Monitoring any temperature rise can help you manage illness.

Measure temperature with Medtech's Digital Thermometer.

KEY FEATURES

- Digital Display
- Flip Top Box
- Accurate Readings
- Automatic Shut-Down
- Memory
- Beep Sound Signal

BENEFITS

- Safe
- Dependable Accuracy
- Fast and Convenient

One unit, for your entire family.



Useful For -

Measuring body temperature orally, rectally, or auxillary.

Do not use the same thermometer for oral and rectal measures.

HOW TO USE

Press the Power-On button. After the beep, the screen will display the right figure.


188.8 °E

The screen will then display the last measured temperature.

36.8 °C OR 96.3 °F

When it is ready to measure the temperature, place the thermometer in the correct place after 'C' or 'F'

LO * OR LO *

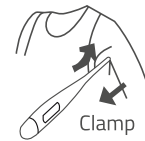

Clamp

BODY TEMPERATURE








37

98.599998

Celsius °

Fahrenheit (F)

Normal Body Temperature Range Table

		 <div>0-2 Years</div>	 <div>3-10 Years</div>	 <div>11-65 Years</div>	 <div>- 65 Years</div>
 <div>Oral</div>	F	—	95.9 - 99.5	97.6 - 99.6	96.4 - 98.5
	C	—	35.5 - 37.5	36.4 - 37.6	35.8 - 36.9
 <div>Rectal</div>	F	97.9 - 100.4	97.9 - 100.4	99.6 - 100.6	97.1 - 99.2
	C	36.6 - 38.0	36.6 - 38.0	37.0 - 38.1	36.2 - 37.3
 <div>Auxillary</div>	F	94.5 - 99.1	96.6 - 98.0	95.3 - 98.4	96.0 - 97.4
	C	34.7 - 37.3	35.9 - 36.7	35.2 - 36.9	35.6 - 36.3

DISCLAIMER : The Normal Body Temperature Range Table is for reference use only. Data provided are derived from multiple sources with varying levels of accuracy. The Developer disclaims all responsibility for the accuracy or completeness of the data shown hereon.

Note: The information provided is best to our knowledge and subject to change without notice.

Picture for illustration purposes only, actual product and accessories may vary. Specifications are subject to change without notice.

TIPS FOR CHANGING THE BATTERY



Figure 1

Replace battery when appears in the lower right corner of LCD display.



Figure 2

Pull battery cover off



Figure 3

Gently pull out plastic circuit board with battery chamber approximately 1 cm (slightly less than 1/2")



Figure 4

Use pointed object such as a pen to remove old battery lawfully. Replace with new 1.5V DC button type Lr41 or Sr41, UCC392, or equivalent. Be sure battery is installed with polarity facing up.



Figure 5

Slide battery chamber place and attach cover.

OUR PRODUCT RANGE

Nebulizer | BP Monitor (Digital/LED) | Steam Inhaler | Digital Thermometer | Pulse Oximeter | Oxygen Concentrator
Air Pressure Mattress | Auto CPAP | Needle Burner & Syringe Destroyer | Bipap | Infrared Thermometer

MEDTECH LIFE PVT. LTD. (Formerly Nulife Global Medical Devices Pvt. Ltd.)

B6, Byculla Service Industries, D.K. Marg, Byculla, Mumbai 400 027, India.

+9122 2374 8372 / 73 +9122 6657 8989 info@medtechlife.com

Domestic: +91 82917 26884 / +91 82917 26885 / 86

Export: +91 91369 23382 / +91 93263 56572