

HANDYPAD

HEATING PAD

HP-01

Using MEDTECH'S
heating pad can
help ease aches
and pains.



KEY FEATURES



Thermostatic
temperature controller



Auto-cut to avoid
over-heating



Three heating levels for
comfort and convenience



Low power Consumption



Made from soft
Velvet Material



Convenient to use

Specification

Product	: Heating Pad
Model No.	: HP-01
Controller	: Yes
Illuminated Indicator	: Yes
Temperature Setting	: 3 setting (Low, Medium, High)
Maximum Temperature	: 80°C
Heating Pad Size	: 33.5 cm x 25 cm
Power Consumption	: 38 watt max
Washable Outer Cover	: Yes
Washable Heating Pad	: No
Cable Length	: 3 Meters
Power Requirement	: 220-240 VAC

IMPORTANT SAFETY INSTRUCTION :

NOTE: To reduce risks of Burns, Electric Shock, Fire, and Personal Injury, This product must be used in Accordance with the following instructions

- Do not use if wet.
- Do not use while sleeping.
- Do not use the cord as a handle.
- Do not use in an Oxygen atmosphere.
- Do not use on areas of insensitive skin.
- Do not use Pins or other metallic means to fasten this Pad in Place.
- Do not use on an infant or on an Animal. To Fasten this Pad in Place.
- Do not use this Pad with liniment, salve or ointment preparation, especially ones that contain Heat producing Ingredients. Skin burns could result.
- Do not tamper with this Pad in any way.
- Do not bunch, sit on or against or crush Pad- Avoid sharp folds.
- Do not bend or pinch cord and Do not use if cord is damaged.
- This Pad is not to be used on by an invalid, a sleeping or unconscious Person,
- A Person with diabetes or A Person with poor Blood circulation.
- Burns can occur regardless of control setting check skin under Pad frequently.
- This heating pad is not designed for use in hospitals.
- Never pull this pad by the supply cord.
- This unit should not be used by children without Adult supervision.
- Place Pad on Top of and not under the part of the body needing heat. Unplug when not in use.



OUR PRODUCT RANGE

Nebulizer | BP Monitor (Digital/LED) | Steam Inhaler | Digital Thermometer | Pulse Oximeter | Oxygen Concentrator
Air Pressure Mattress | Auto CPAP | Needle Burner & Syringe Destroyer | Bipap | Infrared Thermometer

MEDTECH LIFE PVT. LTD. (Formerly Nulife Global Medical Devices Pvt. Ltd.)

Regd. address: B6, Byculla Service Industries, Off D. K. Marg, Byculla, Mumbai-400 027, Maharashtra, INDIA.

+9122 2374 8372 / 73 +9122 6657 8989 info@medtechlife.com

Domestic: +91 82917 26884 / +91 82917 26885 / 86

Export: +91 91369 23382 / +91 93263 56572