

## Range of Electromedical Devices

**MEDTECH**   
Caring for life



**NEBULIZER - HANDYNEB**  
**CLASSIC**



**NEBULIZER - HANDYNEB**  
**SMART**



**NEBULIZER - HANDYNEB**  
**PRO**



**BP MONITOR - NOVACHECK**  
**BP-18**



**BP MONITOR - NOVACHECK**  
**BP-11**



**BP MONITOR - NOVACHECK**  
**BP-11 BL**



**STEAM INHALER - HANDYVAP**  
**VAP-01**



**PULSE OXIMETER - OXYGARD**  
**OG-05**



**DIGITAL THERMOMETER - HANDY**  
**TMP-02**



**HEATING PAD - HANDYPAD**  
**HP-01**



**MASSAGER - GUN**  
**GMV-4**



**MASSAGER - GUN**  
**MPV-1**

**MEDTECH**   
Caring for life

# Nebulizer Kit

## NebuKit®


**3X**



80 mm (W)

### Insist on

#### TWO PARTS BOTTLE

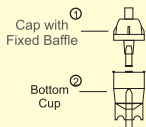

Two Parts Bottle

Because of fix baffle,  
it is Safe for nebulization.  
No fear of losing baffle

#### THREE PARTS BOTTLE

##### Conventional Bottle



Three Parts Bottle

It is easy to loose  
Baffle as it is not fixed  
So if baffle falls Nebulization  
cannot take place and kit is non  
functional

### RESPIRATORY THERAPY PRODUCT

- Disposable for Single Patient Use
- Non Intended for Reprocessing

#### Features:

- Non-Toxic PVC Tube and mask
- 2 Meter PVC Tube
- BPA-free, 2-Part Medicine Cup

#### ⚠ CAUTION:

- Ensure that all connection are secured and air circulates freely through the tubing.
- Do not store at extreme temperature and humidity.  
Store in cool & dry place.

- Do not attach the outlet of the nebulizer directly to an endotracheal tube, tracheostomy tube or other device that does not offer an alternate pathway for exhalation.

#### INSTRUCTION FOR USE:

These directions are general guidelines intended for use by qualified medical personnel.

1. Place prescribed solution into the bottom cup of nebulizer bottle.
2. Attach cap to nebulizer cup with a clockwise rotation.
3. Attach the tubing between the gas source and the nebulizer.
4. Connect the nebulizer outlet on top of nebulizer cap to the mask inlet.
5. Start gas flow Check for aerosol mist; if necessary, tap the device until it begins to nebulize.
6. Place the mask on the patient's face with the elastic strap below the ears and around the neck. Gently pull the ends of the strap until the mask is secured. Shape the metal strap on the mask to fit the face.
7. During treatment, periodically tap nebulizer to minimize residual volume.

Mfg. By

**MEDTECH LIFE PRIVATE LIMITED**

Regd. address : B6,Byculla Service  
Industries, D.K.Marg, Sussex Road, Byculla (East),  
Mumbai-400027, Maharashtra, INDIA.  
Customer Care No.+917208088720  
E-Mail:support@medtechlife.com  
Website : www.medtechlife.com

P02627

Check out our  
wide range of  
products



AR01416-00-00

120 mm (L)

# This Page Not to be Print

|  |   |  |
|--|---|--|
| <div>Prepared by<br/>RUPALI</div> <div></div> | <div>Verified by<br/>LEKHRAJ</div> <div></div> | <div>Approved by<br/>BHOMRAJ GIRI</div> <div></div> |
|--|---|--|

## REVISION TABLE:

| Item Code: P02627 |                  | ARTWORK NO.: AR01416-00-00 |            |          |            |
|-------------------|------------------|----------------------------|------------|----------|------------|
| Sr. No.           | Primary Revision | Secondary Revision         | Date       | Location | Details    |
| 1.                | 00               | 00                         | 29-09-2023 | -        | New Design |