


**Personal Scale
WS05**

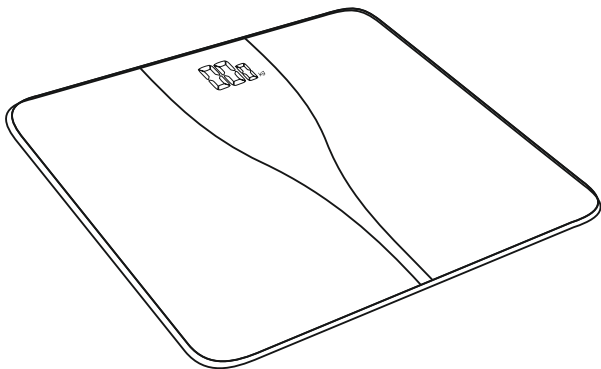

**INSTRUCTION MANUAL
&
WARRANTY CARD**

Please thoroughly read the instructions manual before using the unit.
Please save this instruction manual for further reference.

PRODUCT SPECIFICATION

- | | | |
|--|-------------------------|--------------------------|
| 1. Four High precision strain gauge sensor | 2. Unit:kg/lb/st:lb | 3. 6mm Tempered glass |
| 4. Max Capacity:180kg/396lbs | 5. Auto on & Auto off | 6. Temperature indicator |
| 7. Low power & Overload indication | 8. Division:0.1kg/0.2lb | |

POWER SUPPLY

Battery: 2x1.5V AAA

Please install AAA battery before you use the scale.

OPERATION





Step 1: Use the scale on the flat, level surface. Do not use on carpet as it may affect accuracy.

Step 2: Step onto the platform, once the weight has stabilized, body weight will be locked on the display.

Step 3: The scale will turn off automatically after 10 seconds of no use.

ERROR CODES (OVERLOAD AND LOW POWER INDICATION)

	1. LO-When the LCD indicate "LO", Please open the battery cover and replace the new battery with correct polarity and negative.
	2. OL-Maximum capacity exceeded. when the scale is overloaded (over 180kg/396lbs) "OL" will appear on the screen.

ADVICE FOR USE & CARE

- Please place the scale on a hard & flat surface when in use to ensure the accuracy of the reading.
- Clean the scale with a wet cloth, but do not allow water to seep into the unit and body scale.
- Please remove the battery from the scale if it has not been used for a long period.
- Do not stack anything on the scale during non use.
- Never use abrasive cleansers or chemical solvents, as they may damage the casing.
- Do not put the scale on upright position, only in the appropriate flat position.
- Do not drop or beat the scale, this may damage the sensor and glass.
- To ensure the life of the scale, please do not put in a wet or hot environment.
- If the scale does not work, please check the battery and ensure the battery is with correct polarity. Please check if the battery needs to be changed when the scale has been used regularly, replace battery and try to start again.
- To switch weight unit exchange display between kilograms (kg) and pounds (lb), please press "Set/Unit" button located on the bottom of the scale.

WARRANTY CARD

Medtech Personal Scale WS05 is made under strict quality control & utmost care is taken to ensure long, trouble free service.

- This unit is warranted for a period of 12 months from the date of purchase.
- This card & relevant invoice must be preserved and presented to the concerned authority when the unit is sent for repairs.
- We undertake to repair or replace at our discretion any part which was originally defective, provided the unit was not tampered with or mishandled.
- The company offers a free pickup and drop service for product, within its warranty period to the service centre.
- The warranty is only against manufacturing defects & shall not be applicable in case of damage due to accident, acts of nature, loss of components, intentional damage, improper installation or if it was opened or serviced by unauthorized persons.
- The warranty is void, if the Product is not operated according to instructions given in the "Instruction Booklet", If any repair work is carried out by persons other than authorized company personnel, or defects are caused by improper or reckless use, which shall be determined by company personnel.
- The warranty shall not extend towards the payment of compensation to anyone or entitle the customers to replacement of the products as a whole.
- The warranty expressly excludes every condition or warranty whether statutory or otherwise, or any claim for consequential loss or damage.
- Warranty does not cover accessories external to the equipment, defects caused due to causes beyond control like lighting, abnormal voltage, acts of God or while in transit to service center or to purchasers/premises.
- In the event of a defect being noticed in the equipment during the warranty period, the company or its authorized customer care center/service dealer reserves the right to retain any parts / or components to replace/repair at its discretion.

Imp & Mkt by:

MEDTECH LIFE PVT. LTD.

Address: Plot no 05, Morai Industrial Park, S.No.315/5, Morai, Valsad,
Gujarat - 396185, India. Made in PRC.

For Consumer complaint, Please contact Consumer Care Officer at above address

☎/📠 +91 720 8088 720 📧 support@medtechlife.com 🌐 www.medtechlife.com



PERSONAL SCALE WS05

Serial No.

Invoice No. Date:

**Stamp and Signature
of Dealer**

**Name and Address
of Dealer**

BEST IN CLASS AFTER SALES SERVICE

Scan QR code for
Repair & Service
within India only

Free **Free**
Pickup Platform

*Terms & Conditions apply



Problem with your Device?
Don't Worry, Just scan the
QR code and fill the form

1

Our delivery partner will
pickup the device
from your home

2

Your device reaches our
factory and we repair/replace
the device within 24 hours

3

Your device comes back to
you right at your doorstep in
perfect condition!

4